ProBase® Cold

[en] Instructions for Use

Self-curing denture base material

Rx ONLY

CE 0123

Manufacturer:
Ivoclar Vivadent AG
Bendererstrasse 2
9494 Schaan/Liechtenstein
www.ivoclar.com

Complies with ISO 20795-1;
EN ISO 20795-1

Date information prepared:
2023-01-26 / Rev. 0

ivoclar

English

1 Intended Use

Intended purpose

- Fabrication of bases for removable dentures
- Fabrication of hard occlusal splints
- Fabrication of orthodontic appliances

Patient target group

- Patients with permanent teeth
- Adult patients with dental implants
- Edentulous adult patients

Intended users / Special training

- Dental laboratory technicians (fabrication of restorations in the dental laboratory)
 - Denturists (fabrication of restorations in the dental laboratory)
- No special training required.

Application

For dental use only.

Description

Self-curing denture base material for the pouring and the packing technique.

Technical data

Characteristics	Value
Flexural strength	≥ 60 MPa
Flexural modulus	≥ 1500 MPa
Residual MMA content	≤ 4.5%
Water absorption (7 days)	≤ 32 µg/mm ³
Solubility (7 days)	≤ 8.0 µg/mm ³

Indications

- Partial edentulism in the anterior and posterior region
- Complete edentulism
- Dental dysfunctions and parafunctions
- Teeth misalignments

Areas of application

- Complete denture prosthetics
- Partial denture prosthetics
- Implant prosthetics
- Relines
- Repairs
- Occlusal splints
- Orthodontic appliances

Contraindications

The use of the product is contraindicated if the patient is known to be allergic to any of its ingredients.

Limitations of use

Direct contact of uncured material with intraoral tissues.

- The ProBase Hot/Cold shade guide cannot be sterilized and is thus not suitable for use on the patient.
 - High heat generation (exceeding 110 °C / 230 °F) should be avoided during the grinding, polishing and cleaning process.
- Avoid contact with solvents.

Processing restrictions

The product is not intended to be reprocessed or reused.

Side effects

In individual cases, local allergic reactions to polymethyl methacrylate materials have been reported.

Interactions

There are no known interactions to date.

Clinical benefit

- Reconstruction of chewing function
- Restoration of esthetics

Composition

- **ProBase Cold polymer**
Polymethyl methacrylate, tributyl o-acetylacrylate, dibenzoyl peroxide
- **ProBase Cold monomer**
Methyl methacrylate, 1,4-butanediol dimethacrylate

2 Application

Application notes

- Make sure that the precast and model have been soaked well in water.
- Apply two layers of the alginate-based isolation agent "Separating Fluid" from Ivoclar and allow to dry thoroughly.
- Roughen the teeth well and wet them with monomer to ensure a sound bond with the denture base.
- The flow and modelling phase are dependent on the chosen mixing ratio and the ambient or storage temperature of the materials.
- The denture base material must be thoroughly mixed using a hard spatula (approx. 20-30 seconds).
- Avoid fast cooling in cold water (tension cracks).
- In order to select the appropriate shade of ProBase Cold, the ProBase Hot/Cold shade guide is available to the user.
- We do not recommend using a gloss varnish.
- The products are intended to be used only by professionals.
- Avoid mixing of different materials.

Pouring technique

1. Preparation

Isolate boiled-out, water-saturated plaster surfaces with two layers of the alginate-based isolation agent "Separating Fluid" and allow to dry thoroughly. Roughen the teeth well and wet them with monomer to ensure a sound bond with the denture base.

2. Dosage

Ideal mixing ratio for one denture: 15 g polymer : 10 ml monomer

3. Mixing

Mix together powder and liquid with a spatula in the given mixing ratio and then blend thoroughly for 20-30 seconds. Leave to stand for 15 seconds so that any air bubbles can escape. For this procedure, a mixing cup and mixing spatula are provided.

4. Flow phase

The flow phase lasts 2.5-3 minutes at room temperature (18-25 °C / 64-77 °F). Pour the material into the precast within this time span.

5. Modelling phase

After a transition period of approx. 4 minutes, the material will assume a firmer consistency and can be modelled for a period of 3 minutes. A higher room temperature shortens the flow and modelling time. The resin material must be placed in the pressure pot within no more than 8-9 minutes.

6. Polymerization

Polymerization takes place in a pressure pot at a temperature of 40 °C / 104 °F and a pressure of 2 bar for 15 minutes (please check).

7. Finishing

Remove the precast, finish and polish in the usual manner.

Packing technique

1. Preparation

Isolate boiled-out, water-saturated plaster surfaces with two layers of the alginate-based isolation agent "Separating Fluid" and allow to dry thoroughly. Roughen the teeth well and wet them with monomer to ensure a sound bond with the denture base.

2. Dosage

Ideal mixing ratio for one denture: 20.5 g polymer : 10 ml monomer

3. Mixing

Mix together powder and liquid with a spatula in the given mixing ratio and then blend thoroughly for 20–30 seconds. Allow the dough to mature in a closed mixing cup (at a temperature of 23 °C / 73 °F) for approx. 3–4 minutes. Subsequently, the dough can be worked for approx. 2 minutes. A higher room temperature shortens the working and the setting time.

4. Pressing

Slightly overfill the hand-warm, wetted and isolated flask halves with resin dough. Carefully close the flask and load with 80 bar pressure in a pressing device. Fix with a clamp.

5. Polymerization

Polymerization takes place with the clamp in place or in the pressing device under constant pressure for 30 minutes (at a room temperature of 23 °C / 73 °F).

6. Deflasking and finishing

Open the flask and remove the plaster. Check the occlusion of the denture and finish in the usual manner.

With both processing techniques, the residual monomer content after polymerizing the material according to the method described is $\leq 4.5\%$

Repair and adjustment possibilities with ProBase Cold

Repairs and adjustments of ProBase Hot, ProBase Cold, SR Ivoclar and Ivobase may be carried out with ProBase Cold by using the pouring technique. Thoroughly roughen the corresponding surfaces and wet with monomer.

3 Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: www.ivoclar.com, and your responsible competent authority.
- The current Instructions for Use are available in the download section of the Ivoclar Vivadent AG website (www.ivoclar.com).

Warnings

- Observe the Safety Data Sheet (SDS) (available in the download section of the Ivoclar Vivadent AG website www.ivoclar.com).
- The monomer contains methyl methacrylate (MMA)
- Methacrylate is easily flammable and irritating (flash point +10 °C / 50 °F).
- Irritating to eyes, skin and respiratory system.
- Wear protective goggles during processing.
- A mask should be worn and a suction device used during grinding.
- Skin contact may cause sensitization.
- Avoid contact of the skin with monomer or uncured material.
- Commercially available medical gloves do not offer protection against the sensitizing effect of methacrylates.
- Do not inhale vapours.
- Keep away from sources of ignition. Do not smoke.
- Do not empty into drains.
- Take precautionary measures against electrostatic discharge.

Disposal information

Remaining stocks must be disposed of according to the corresponding national legal requirements.

Residual risks

Users should be aware that any dental intervention in the oral cavity involves certain risks.

The following clinical residual risks are known:

- Ingestion of fragments
- Occurrence of denture stomatitis

4 Shelf life and storage

- Storage temperature 2–28°C (36–82°F); we recommend storing monomers in the refrigerator.
- Do not use the product after the indicated expiry date.
- Expiry date: See note on the packages.
- Protect the product from direct sunlight.
- Before use, visually inspect the packaging and the product for damage. In case of any doubts, please contact Ivoclar Vivadent AG or your local dealer.

5 Additional information

Keep material out of the reach of children!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the product for its suitability and use for any purpose not explicitly stated in the Instructions. The user is obliged to fully document and store the batch number, so that traceability of the batch is ensured.

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